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## INSTRUCTIONS FOR USE

### Important information:

-  – Read these instructions for use before using for the first time.
- Keep these instructions for use in a safe place.
- Keep the belAir® nasal rinse out of the reach of children.
- Do not use the belAir® nasal rinse for more than 2 months after opening the package. Mark the label on the bottle with the date it was opened.
- After the expiry date, dispose of the belAir® nasal rinse.
- People with mental, physical or sensory disabilities and children do not recognise the risks that may arise when using the belAir® nasal rinse. Never allow anyone from these groups access to the belAir® nasal rinse without supervision. For example, there is a risk that the cap may be swallowed and block the airway, or that the concentrate may be used undiluted.
- If rinse solution gets into the eyes, flush the eyes with water.
- belAir® nasal rinse with dexpanthenol is a saturated salt solution (25% - 27%) and is not intended for ingestion.
- Use the belAir® nasal rinse only in diluted form.
- Only use the belAir® nasal rinse together with the INQUA® or belAir® nasal douche.
- Follow the instructions for use for the INQUA® or belAir® nasal douche as well.

### Contents:

Natural saline solution	98.8%
Aqueous dexpanthenol sol. (75%)	1.2%
Free of preservatives.	

### Contents:

250 ml concentrated solution for 30 rinses

### Areas of use:

belAir® nasal rinse, supplemented with the soothing effect of dexpanthenol, is used in diluted, isotonic form for thorough rinsing and cleaning of the nasal mucus. The nasal rinse assists in removing germs, dust and contaminants from the nose. This is particularly useful in dusty atmospheres or when pollen counts are high if the user suffers from hay fever.

Nasal rinses are also used in dry atmospheres – e.g., in heated or air conditioned rooms – to moisten the nasal mucous membrane. They can be used to loosen secretions in stuffy noses. A nasal rinse also helps in the treatment of inflamed paranasal sinuses and cold prevention.

### Limitations:

The belAir® nasal rinse must not be used in cases of

- injury to the nasal roof, the paranasal sinus walls or the base of the skull
- palatine paralysis
- nosebleeds
- a predisposition to choking
- individual intolerance or hypersensitivity towards any of the ingredients in the saline solution listed above

### Side effects:

Correct use may cause mild irritation, such as a ticklish or stinging sensation in the nose, and an individual reaction (headache and more abundant mucus production). If you are using topical cold medicines (e.g., decongestant nose drops) at the same time, allow a reasonable pause between the two types of preparation.

No other limitations or cross-reactions with other substances have been reported with correct use.

## Application:

### Note:

Only use the belAir® nasal rinse together with the INQUA® or belAir® nasal douche. The douche was developed specifically for use with the belAir® nasal rinse.

Unscrew the cap on the belAir® nasal rinse supply bottle, but do not remove it from the bottle.

- Hold the bottle upright.
- Gently squeeze the bottle.  
The concentrated saline solution will rise into the upper dosing chamber.
- Keep squeezing until the liquid reaches the dosage mark. When pressure on the bottle wall is released, excess saline solution drains out of the dosing device and runs back into the bottle.
- Take the cap off the supply bottle.
- For reasons of hygiene, avoid contact with the inside of the dosing device.
- Pour the dosed unit into the clean, prepared INQUA® or belAir® nasal douche.
- Close the supply bottle securely with the screw cap.
- Fill the INQUA® or belAir® nasal douche up to the bottom of the screw thread. Use drinking water at a temperature of 37 °C, i.e. normal body temperature.

### Caution:

- Use only drinking water to prepare the nasal rinse solution. People with an elevated risk of infection (e.g., those with a suppressed immune system, people suffering from mucoviscidosis/CF) must use boiled or sterile water.
- Do not attempt to keep unused residue of the rinse solution. Dispose of all unused solution. Otherwise bacterial contamination may occur.  
Rinse solutions must always be prepared fresh.

The rest of the procedure is described in the instructions for use of the INQUA® or belAir® nasal douche.

## Frequency of use:

You can rinse the nose as needed up to 3 times a day with 250 ml isotonic saline solution. The recommended frequency is once per day. For rinsing and cleaning the nasal passages over a prolonged period, we recommend rinsing 2 - 3 times per week.

If your symptoms have not improved after 14 days, consult your doctor or chemist.

## Storage:

Always keep the bottle closed tightly.  
Protect from sunlight.

## Explanation of symbols:

 Manufacturer	 Store out of the reach of children
 Caution	 Please follow the instructions for use
 Lot no.	 Protect from sunlight
 Use by	 Order number

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