
INQUA[®] Vernebler 1.2

für INQUA[®] Inhalator 1.2

REF BR027100

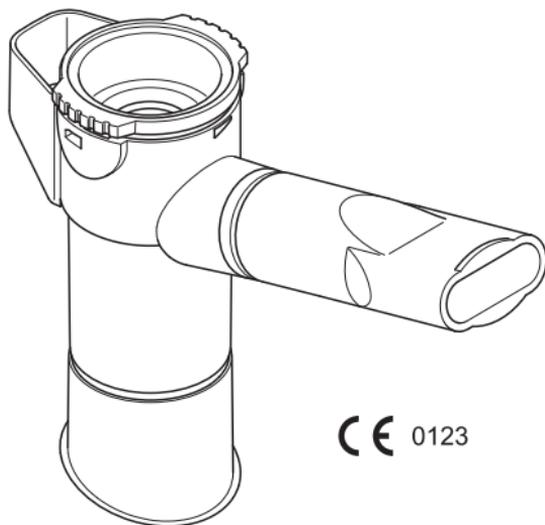CE 0123

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en Instructions for use _____ 39

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INQUA[®] Vernebler 1.2

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Instructions for use

TABLE OF CONTENTS

1	GENERAL	40
1.1	Manufacturer's data.....	40
1.2	Notes on the instructions for use.....	40
2	PRODUCT DESCRIPTION	42
2.1	Components	42
2.2	Intended purpose	43
2.3	Correct use	43
2.4	Conformity	45
2.5	Product combinations.....	45
3	NOTES ON PRODUCT SAFETY	46
4	USE	48
4.1	Preparing for inhalation	49
4.2	Performing the inhalation	52
4.3	Completing the inhalation.....	55
5	HYGIENIC RE-USE	56
5.1	Care of the connection tubing	57
5.2	Preparation for cleaning/disinfection	58
5.3	Cleaning with warm drinking water.....	59
5.4	Disinfection.....	60
5.5	Drying	64
5.6	Storage.....	65
6	MISCELLANEOUS	66
6.1	Storage and transportation conditions.....	66
6.2	Material information.....	66
6.3	Operating life	66
6.4	Disposal.....	67
6.5	Explanation of symbols	67
7	SPARE PARTS/ACCESSORIES	68
8	TECHNICAL DATA	68
8.1	General.....	68
8.2	Aerosol characteristics in accordance with EN 13544-1 appendix CC.....	69

1 GENERAL

WARNING:

- Read these instructions carefully and completely before using for the first time, and keep them in a safe place. Do not use the INQUA[®] nebuliser unless you fully understand the contents of these instructions for use.
If you fail to comply with the instructions for use, injury to health or damage to the product cannot be ruled out.
- If symptoms persist or if your condition worsens, discontinue the application and contact your doctor immediately.

1.1 Manufacturer's data

See back of instructions for use.

1.2 Notes on the instructions for use

Purpose of the instructions for use

These instructions for use describe how the INQUA[®] nebuliser is assembled, how it works, and how it should be used. They are intended to ensure that the individual components can be used in complete safety.

Instructions for using the compressor that will be attached to the nebuliser are not included in these instructions for use. Please read the instructions for use of the compressor which accompany that item.

Representation of hazard warnings

In these instructions for use, hazard warnings are indicated by a warning triangle and a signal word. The following signal words are used depending on the nature and degree of danger being described:

- **NOTE**
Indicates general precautionary measures which are to be observed to avoid damaging the product during use.
- **CAUTION**
Indicates a hazard that may cause mild to moderate injury or damage or may adversely affect the treatment if it is not avoided.
- **WARNING**
Indicates a hazard that may cause serious injury or death if it is not avoided.

Forms of emphasis

The following forms of emphasis are used in these instructions for use to indicate important parts of the text:

- ⇨ The arrow indicates that a given action must be taken.


Note on storing the instructions for use

The instructions for use in printed form are available from INQUA GmbH upon request (see manufacturer information on the back of the instructions for use), or they can be downloaded for the respective product from www.inqua.de. **Check whether a more recent version of the instructions for use is available at regular intervals.**

2 PRODUCT DESCRIPTION

2.1 Components

INQUA[®] nebuliser set 1.2 (applied part)

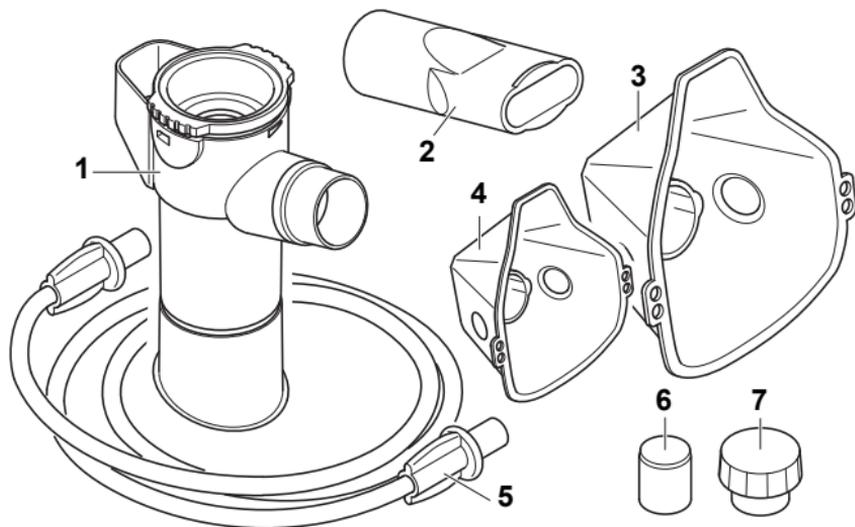- | | |
|--------------------------|---|
| 1 Nebuliser (pre-fitted) | 5 Connection tubing |
| 2 Mouthpiece | 6 Air filter for the INQUA [®] compressor (Type 430), for use see instructions for use of the compressor |
| 3 Adult mask | 7 Removal tool for changing the filter, for use see instructions for use of the compressor |
| 4 Child mask | |

2.2 Intended purpose

Together with the compressor being used, the INQUA[®] nebuliser forms an inhalation device for nebuliser treatment of the upper and lower airways with medication aerosol. It is suitable for temporary¹⁾ oral or nasal inhalation. It may be used to inhale all nebuliser solutions that have been authorised for use in aerosol therapy or prescribed or recommended by a doctor. Always read the notes (restrictions, frequency of use) in the accompanying information for the medication to be inhaled, and ask your doctor or pharmacist if you have any questions.

2.3 Correct use

The INQUA[®] nebuliser generates a moist-cool aerosol that cannot be heated up. The nebuliser must only be used for nebuliser treatment. It must not be used in conjunction with anaesthesia breathing systems or in the breathing system of respirators.

The INQUA[®] nebuliser is intended exclusively for use in the home and together with the INQUA[®] compressor. For reasons of hygiene, it must only be used by one person.

The nebuliser is only designed for patients who are able to breathe by themselves and are conscious. This is the only way to ensure effective treatment and avoid a choking hazard.

The nebuliser equipment is suitable for use by children aged three and older up to adult. The nebuliser system can be used for nebuliser treatment of the **lower airways** (bronchi), if the patient uses the **mouthpiece**, breathing in through the mouth and out through the nose. The mask should only be used for

1) *Each use lasts less than 1 hour according to the European Medical Device Directive 93/42/EEC, in this case max. 20 minutes.*

treatment of the lower airways in exceptional circumstances. In this case, the patient must be sure to breathe in through the mouth.

The nebuliser system can be used for nebuliser treatment of the **upper airways** if the patient is a child using the accompanying **child mask** and breathing in and out through the nose. Adults use the **adult mask**, breathing in and out through the nose.

Nebuliser treatment for babies: In order to ensure efficient nebuliser treatment for the lower airways in babies (up to age three), you need a special baby nebuliser. In this case, contact the manufacturer (for address see the back of the instructions for use).

Nebuliser therapy for patients with a surgically created opening in the windpipe (tracheostoma): The nebuliser system is **not** suitable for patients who have undergone a tracheostomy. Tracheostomised patients breathe through a surgically created opening in the windpipe (tracheostoma). It is not possible to assure effective treatment for this clinical picture with this equipment.

Note that the connection tubing included with the nebuliser kit must only be used for the inhalation device. If multiple tubing systems are used at the same time for different applications (e.g., infusion and inhalation), there is a risk that the wrong tubing systems may be used by mistake, which in extreme cases may result in death.

Any application other than the use described here is considered improper, and therefore dangerous. In all cases the user is liable for the safe operation of the nebuliser if it is not used according to the instructions.

Contraindications

There are no contraindications known to INQUA GmbH.

2.4 Conformity

The INQUA[®] nebuliser satisfies the requirements of Directive 93/42/EEC (EC Medical Device Directive).

2.5 Product combinations

Use the INQUA[®] nebuliser exclusively with the compressors recommended by INQUA[®].

3 NOTES ON PRODUCT SAFETY

The following general instructions apply:

- People with mental, physical or sensory disabilities and children do not recognise the risks that may arise when using the inhalation device and its components. Never allow anyone from these groups access to the inhalation device without supervision. For example, there is a risk that individual parts may be swallowed and block the airways, or bitten or bitten off, or that the inhalation device may be pulled off the table by the connection tubing or power cord. There is also a risk of strangulation with the connection tubing or the power cord. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.
- You must conscientiously follow the general hygiene procedures when using your nebuliser, e.g., thorough washing of hands or disinfection if required (see chapter "HYGIENIC RE-USE", page 56) before starting the inhalation session.
- Use only liquid medications that have been approved for use in nebuliser therapy with a compressed air inhalation device in your INQUA[®] nebuliser. Ask your doctor or pharmacist about the selection and use of the medications available.
- The following are generally unsuitable for use in inhalation devices: cough mixtures, gargles, balsamic preparations, oils of medicinal plants, and drops that are used as ointments or added to the bath. Inhaling these medications can cause irreversible damage to the airways. Such substances can also impair the function of the inhalation device permanently.

- Use only accessories and spare parts that are recommended by INQUA in order to obtain the best possible treatment (see Chapter "SPARE PARTS/ACCESSORIES", page 68).
- Note the required ambient conditions for operation:

Ambient temperature:	+10 °C to +30 °C
Relative humidity:	30% to 75% (non-condensing)
Atmospheric pressure:	700 hPa to 1,060 hPa

- INQUA GmbH cannot be held responsible for damage that is caused by improper or incorrect use, e.g., use of a compressor made by another manufacturer. Therefore, you are advised to read the instructions for use of the INQUA[®] nebuliser **and** of the INQUA[®] compressor carefully and to familiarise yourself with the device before using it.

4 USE

WARNING: Danger to health from improper use

Small parts can be swallowed or block the airways, thereby posing a risk of suffocation. Therefore, infants and anyone who requires assistance must remain under constant supervision during the nebuliser treatment.

WARNING: Danger of infection from contaminated nebuliser

Follow the hygiene regulations before each use and after any prolonged interruption in use. Make sure that the nebuliser has been cleaned and disinfected before it is used for the first time or after any previous treatment (see Chapter "HYGIENIC RE-USE", page 56).

CAUTION: Impaired treatment due to damaged parts

Inspect all nebuliser components and accessories before every use. Replace any broken, misshapen or seriously discoloured parts. Please also follow the instructions for assembly given below. Damaged components and/or an incorrectly assembled nebuliser may impair **functioning of the nebuliser** and thus **treatment as well**.


WARNING: Risk of infection due to cross-contamination

If the nebuliser is used for more than one patient, there is a risk that **germs** will be **transmitted** from one patient to the next via the nebuliser and its accessories. To avoid this, the nebuliser and the accessories must be replaced.

4.1 Preparing for inhalation

Assembling the nebuliser

If the nebuliser has been dismantled, follow the instructions below to assemble it for use: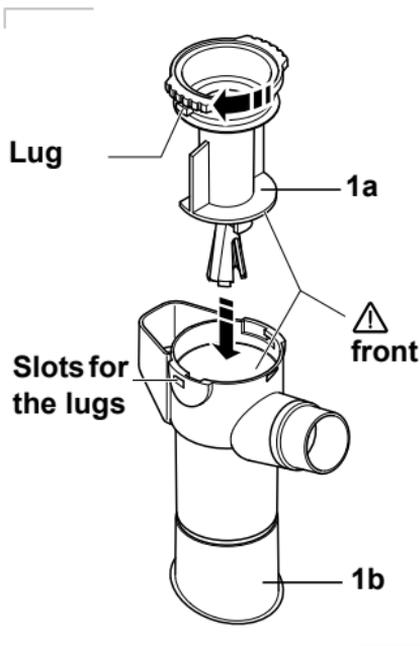


Note that the nebuliser upper part (**1a**) and the nebuliser lower part (**1b**) are attached to each other by a "bayonet" closure.

⇒ Insert the nebuliser upper section carefully in the nebuliser lower section.

⇒ Pay particular attention to the note in the **front** of the drawing.

The lugs in the nebuliser upper part are different sizes and only fit in the different sized slots in the nebuliser lower part in a certain direction.

Twist the nebuliser upper part clockwise while pressing gently to engage the lugs on the nebuliser upper part in the slots in the nebuliser lower part. The two nebuliser parts will be attached securely by gently twisting clockwise.

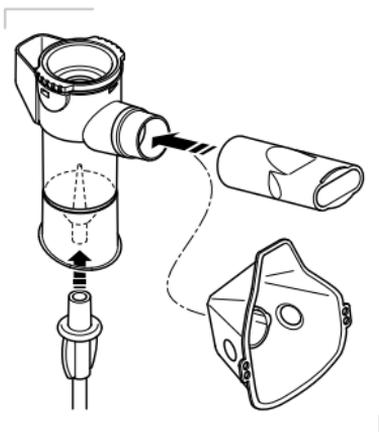⇒ Connect one end of the connection tubing to the air outlet on the INQUA[®] compressor (see the instructions for use of the INQUA[®] compressor) and the other end to the INQUA[®] nebuliser.

Inhalation for the lower airways:

⇒ Attach the mouthpiece or child mask to the nebuliser. The child mask should only be used in exceptional cases.

Inhalation for the upper airways:

⇒ Attach the mask to the nebuliser.

Filling with inhalation solution

⇒ Insert the nebuliser in the holder on the compressor intended for this purpose.

⇒ Pour the quantity of inhalation solution prescribed by your doctor or the manufacturer into the top of the nebuliser.

Make sure that the liquid added does not pass the top scale graduation mark (max. fill quantity 8 ml). Overfilling can greatly impair the nebulisation of medication.

The nebuliser is ready for use.

4.2 Performing the inhalation

WARNING: Danger to health if connection tubings are mixed up

Check carefully to ensure that the other end of the connection tubing connected to the compressor is attached to the nebuliser.

If multiple tubing systems are used at the same time for different applications (e.g., infusion and inhalation), there is a risk that the wrong tubing systems may be used by mistake, which in extreme cases may result in death.

CAUTION: Inadequate medication nebulising due to incorrect assembly

Before the patient starts inhaling, ensure that all parts are firmly connected to each other. Make sure that the connection tubing is fitted securely on the air outlet on the INQUA[®] compressor and on the nebuliser.

CAUTION: Inadequate delivery of medication because nebuliser held incorrectly

Always hold the nebuliser upright during inhalation.

Inhalation for the lower airways

Inhalation with the mouthpiece is the most effective form of inhalation for the lower airways since it results in minimum medication loss on the way to the lung. The child mask is only recommended for children if they are unable to inhale using the mouthpiece.

When a mask is used, the patient must always breathe in through the mouth during the session.

- ⇒ Sit in an upright position and relax.
- ⇒ Take the nebuliser out of its holder on the compressor and hold it upright.
- ⇒ Switch the compressor on.
The nebuliser will generate a visible mist (aerosol).

- ⇒ Hold the mouthpiece between your teeth and enclose it with your lips.

- ⇒ Breathe in through the mouthpiece as calmly, deeply and evenly as possible.

- ⇒ Breathe out calmly through your nose. When breathing out through the nose, a small quantity of mist (aerosol) may escape from the top of the nebuliser. These losses are not significant.

Inhalation for the upper airways

- ⇒ Hold the nebuliser vertically, sit in a relaxed, upright position.
- ⇒ Switch the compressor on.
The nebuliser will generate a visible mist (aerosol).

When using the mask:

- ⇒ Hold the mask up to the face so that it forms an airtight seal over the mouth and nose.
- ⇒ Breathe in calmly, deeply and evenly through the nose, and after a few seconds breathe out again through the nose.

4.3 Completing the inhalation

The duration of inhalation depends on the quantity of medication used and the breathing pattern.

Inhalation is complete when no more aerosol is produced by the nebuliser, or the aerosol only emerges from the nebuliser intermittently. The noise made by the nebuliser will change.

A small quantity of medication (approx. 1 ml) will be left in the nebuliser even after an inhalation session.

- ⇒ When the inhalation session has ended, turn the INQUA[®] compressor off at the On/Off switch.
- ⇒ Pull the power plug out of the socket (see instructions for use of the INQUA[®] compressor) and disconnect the nebuliser from the connection tubing.

If the nebuliser is to be used for several different medications one after the other in one inhalation session, the nebuliser must be cleaned before it is filled with different medication (see Chapter "HYGIENIC RE-USE", Section "Cleaning with warm drinking water", page 59).

WARNING: Bacterial contamination caused by residues

Bacteria can breed very quickly in nebulisers, mouth-pieces or masks that have not been cleaned properly and in liquids that are left in the nebuliser, thereby endangering health, for example by causing infections.

You should therefore begin preparation for hygienic re-use as soon as the inhalation session is ended. Always empty out the residue in the nebuliser and never store any liquids in it.

5 HYGIENIC RE-USE

WARNING:

In order to prevent a risk to health, e.g., infection due to a contaminated nebuliser, it is imperative that the following hygiene regulations are followed.

The INQUA[®] nebuliser is designed for multiple uses. It must not be used by more than one person. A dedicated nebuliser, mouthpiece and/or mask must be provided for exclusive use by each individual patient.

To do this, follow these instructions:

- The nebuliser and the mouthpiece or respective mask must be cleaned **and** disinfected before they are used for the first time and after each use.
- Wash your hands thoroughly before every use and cleaning. Additional requirements concerning the necessary hygienic preparations (hand disinfection, handling of medications and inhalation solutions) in high-risk groups (e.g., users with suppressed immune systems, patients with cystic fibrosis) can be obtained from the respective self-help groups.
- The connection tubing cannot be cleaned nor disinfected either chemically or by boiling (see section "Care of the connection tubing", page 57).
- Ensure that the INQUA[®] nebuliser is dried thoroughly after each cleaning and disinfection. Condensation or residual wetness can present an increased risk through the growth of bacteria.
- Please read Section "Material information" in Chapter "MISCELLANEOUS" on page 66.

- Check all the parts of your nebuliser and accessories regularly and replace any defective (broken, misshapen, discoloured) parts (see Chapter "SPARE PARTS/ACCESSORIES", page 68).
- Please also read and follow the hygiene instructions in the instructions for use of the INQUA[®] compressor.

Like all plastic parts, the INQUA[®] nebuliser is susceptible to a certain degree of wear when it is used and reprocessed frequently (wearing part). Over time, this can cause a change in the aerosol, which in turn impairs the therapeutic efficiency.

You should replace the nebuliser at least once a year. Replacement sets can be obtained from your sales partner or from INQUA GmbH (see Chapter "SPARE PARTS/ACCESSORIES", page 68).

Proof of suitability of the INQUA[®] nebuliser for effective cleaning and disinfection has been obtained through an independent testing laboratory using the procedures recommended starting on page 59. Using the named alternatives remains the responsibility of the user.

5.1 Care of the connection tubing

WARNING: Bacterial contamination caused by residual moisture

Any residual moisture in the connection tubing can present a risk to health, e.g., by causing an infection.

- ⇒ After inhalation, check the connection tubing for condensation. Detach the nebuliser from the connection tubing. Plug the compressor into the socket and switch the compressor on. Allow the compressor to continue running until the air-stream has removed all the condensation in the connection


tubing. Then, switch the base unit off, unplug the power plug from the socket, and disconnect the connection tubing from the compressor.

- ⇒ Do not use water or any other cleaning or disinfecting solutions to clean the connection tubing.
Replace the connection tubing if it is contaminated.

5.2 Preparation for cleaning/disinfection


Directly after each treatment, all parts of the INQUA[®] nebuliser must be cleaned of medication residue and contamination.

In order to help reduce water pollution, do not dispose of the medication residue in the waste water system. Wipe it up with tissues and dispose of the tissues with non-recyclable waste.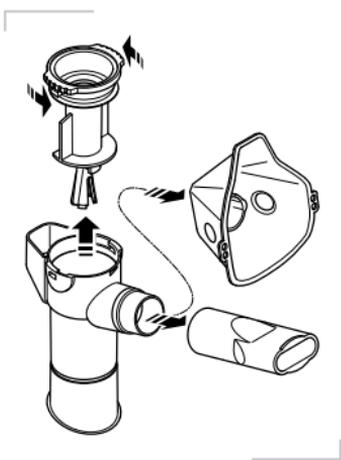

- ⇒ If this has not been done, disconnect the connection tubing from the INQUA[®] nebuliser.
- ⇒ Dismantle the nebuliser into its individual parts.
- ⇒ Rinse all components of the nebuliser under running tap water. This must be done especially thoroughly if cleaning and disinfection are not carried out straight away.

5.3 Cleaning with warm drinking water

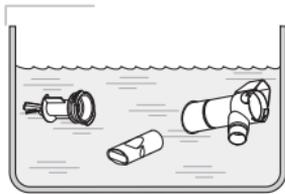- ⇒ Place the disassembled components in warm drinking water with a little dishwashing liquid with no skin protecting or disinfecting additives (dosage according to the manufacturer's recommendations) for about 5 minutes, until the residues of the inhalation solution and other contaminants are removed.
- ⇒ Use a small, clean brush to remove heavy soiling. For hygiene reasons, this brush must be used exclusively for cleaning the nebuliser.
- ⇒ Then rinse all parts thoroughly (for a least 1 minute) in warm running water (approx. 40 °C without dishwashing liquid) to remove all remaining dishwashing liquid.
- ⇒ Pour out the water that has collected in the individual parts. You can remove excess water more quickly by shaking the parts.

5.4 Disinfection

Disinfect the dismantled nebuliser and the mouthpiece or the mask used immediately after cleaning.

CAUTION: Risk of infection due to inadequate disinfection

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection. A nebuliser cannot be disinfected effectively unless it has been cleaned thoroughly. Disinfection is not completed properly until the specified minimum disinfection time has elapsed. Do not switch the disinfectant off until the specified minimum disinfection time has passed.

If you use a microwave for disinfection, ensure that the appliance operates with the necessary wattage [W]. Also make sure that the disinfectants used are clean and functioning properly.

Disinfection with boiling water

⇒ Place the individual parts of your INQUA[®] nebuliser in boiling water for at least 15 minutes.

Use a clean pot and fresh drinking water.

Drinking water contains dissolved particles (e.g., salts) that can form a milky film on plastic parts during the boiling process, depending on their type and quantity. This film can be prevented by using distilled water in the boiling operation.

When boiling, make sure that there is enough water in the pot to prevent the nebuliser parts from coming into contact with the base of the pot.

WARNING: Risk of fire if boiling operation is not supervised!

As soon as the water has evaporated completely, the plastic in the pot will start to melt. If the heat source is not cut off, the plastic will decompose, giving off smoke, and may cause a fire.

Never leave the boiling procedure unattended!

Disinfection with hot steam

The INQUA[®] nebuliser can be disinfected using a steam disinfector (disinfector for baby milk bottles). To ensure that the hot steam passes through all the individual components of the nebuliser, the components must be placed upright in the disinfector. For effective disinfection, use a disinfector with a runtime of at least 6 minutes. Regarding the performance of the disinfection, the duration of the disinfection procedure and the quantity of water required, follow the instructions for use of the disinfector you are using.

Disinfection in the microwave

Disinfectors for standard domestic microwave ovens can be used for effective thermal disinfection of the nebuliser components.

NOTICE:

If the components are heated in the microwave oven without an appropriate disinfector or if the "Grill function" is used, the components will be damaged by exposure to excessive heat.

Therefore, the components must not be exposed to microwave radiation unless they are in a microwave disinfector.

WARNING: Risk of explosion and fire due to use of unsuitable liquids

Use only water for disinfection.

⇒ Place the components in the disinfector with enough space between them. If possible, place the parts in the appliance so that the steam created by the microwave action can flow through the parts to the greatest effect.

Maintain the following times when disinfecting:

Wattage [W]	Disinfection time	Cooling time
850 – 950	4 min.	2 min.
600 – 850	6 min.	2 min.

CAUTION: Risk of burning

At the end of the disinfection process, the parts may still be very hot. Therefore, it is essential to wait for the cooling time to pass before taking the parts out.

Info:

Information about the amount of water required and about the disinfection process is included in the instructions for use of the devices in question (disinfector and microwave).

**CAUTION: Risk of infection due to growth of bacteria**

Inadequate or ineffective disinfection encourages the growth of bacteria and thus increases the risk of infection.

- ⇒ Always comply with the specified minimum disinfection time and the required microwave power.
- ⇒ The disinfectant must be perfectly clean.
- ⇒ Check regularly that the microwave and the disinfectant are functioning perfectly.
- ⇒ Use only water for disinfection, never use chemicals.

NOTICE: Risk of damaging the components with microwave radiation

If the product components are exposed to microwave radiation directly and without protection, they will be damaged.


- ⇒ When heating the components in the microwave, always use a suitable microwave disinfectant.
- ⇒ Do not use the grill function

Chemical disinfection

For chemical disinfection, use the chemical Bomix[®] plus. To ensure safety when handling chemicals, always follow the instructions for use of the Bomix[®] plus, particularly the accompanying safety instructions.

**CAUTION: Risk of infection due to inadequate chemical disinfection**

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection. Adequate chemical disinfection can only be assured if the specified mixing ratio and application time are adhered to, and if all individual parts are completely immersed in the solution for the entire application time. There must not be any air pockets or bubbles.

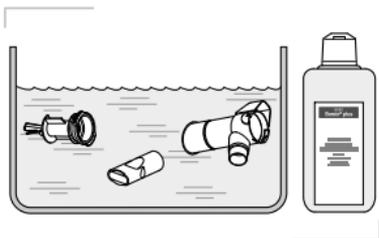⇒ Prepare a 2% solution of Bomix[®] plus by mixing 10 ml of the concentrate with 500 ml tap water.

⇒ Place all the individual parts in the prepared solution and leave them to soak for 5 minutes.

Info:

If the application period is exceeded significantly, the plastic parts may take on the smell of the disinfectant.

⇒ Rinse all parts thoroughly in running water.

CAUTION: Risk of an allergic reaction

Residues of the disinfectant solution can cause allergic reactions or inflammation of the mucous membrane. Therefore, all chemically disinfected parts must be rinsed off thoroughly.

⇒ Dispose of the used solution (the diluted solution can be disposed of down the drain).

5.5 Drying

⇒ After disinfection, place the individual components of the INQUA[®] nebuliser on a dry, clean and absorbent surface. You can remove excess water more quickly by shaking all parts.

⇒ Let them dry completely.

The components should not be dried in damp rooms (e.g., the bathroom).

5.6 Storage

WARNING: Bacterial contamination caused by residual moisture

Inadequate drying can present a risk to health, e.g., by causing an infection.

Make sure that all components are free of residual moisture before reassembling them and/or packing them away or using them for inhalation again. All components must be perfectly dry.

WARNING: Danger to health from choking or misuse

If children operate the inhalation device without supervision, there is a risk that they may swallow small parts, which may block the airways or that incorrect use may cause injury (e.g., strangulation with the connection tubing or power cord).

Keep the INQUA[®] nebuliser out of children's reach.

- ⇒ Check all product components. Replace any broken, misshapen or discoloured parts.
- ⇒ Between uses, and in particular during long breaks in treatment, wrap the dry nebuliser in a dry, clean, lint-free cloth (e.g., a tea-towel) and keep it in a dry, dust-free environment.

6 MISCELLANEOUS

6.1 Storage and transportation conditions

Do not store the INQUA[®] nebuliser in damp rooms (e.g., bathroom). Do not store or transport it together with damp objects. Protect the INQUA[®] nebuliser from continuous, direct sunlight while it is being stored and moved.

In general, the nebuliser should be stored as described in Chapter "HYGIENIC RE-USE", Section "Storage", page 65 between applications. Be sure to observe the following ambient conditions:

Minimum ambient temperature:	-25 °C (without monitoring of relative humidity)
Maximum ambient temperature:	+70 °C (with relative humidity of up to 93%, non-condensing)
Humidity:	0% – 93%
Atmospheric pressure:	500 hPa to 1,060 hPa

6.2 Material information

The nebuliser upper part, the nebuliser lower part, the masks and the mouthpiece are made from polypropylene.

6.3 Operating life

The INQUA[®] nebuliser has an average expected operating life of:

- approx. 300 disinfection cycles
- max. 1 year

6.4 Disposal

The components of the INQUA[®] nebuliser and the replacement set must be disposed of with non-recyclable waste unless other regulations apply in the country where you are located.

6.5 Explanation of symbols

Symbol for manufacturer

Symbol for Follow instructions for use

 0123 The INQUA[®] nebuliser satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.

Symbol for reference number

Symbol for batch number

Symbol for Caution

7 SPARE PARTS/ACCESSORIES

Applied part

Item description	Item no.
INQUA [®] nebuliser set for INQUA [®] Inhalator 1.2	BR027100

8 TECHNICAL DATA

8.1 General

Recommended nebuliser gases:	Air
Minimum flow:	3.0 l/min.
Corresponding minimum operating pressure:	50 kPa (0.5 bar)
Maximum flow:	6.0 l/min.
Corresponding maximum operating pressure:	200 kPa (2.0 bar)
Minimum fill volume:	2 ml
Maximum fill volume:	8 ml


8.2 Aerosol characteristics in accordance with EN 13544-1 appendix CC

The output of the nebuliser depends on the medication being used. Consequently, the following data may not apply for medications that exist as suspensions or are very viscous. If in doubt, ask the manufacturer of the medication concerned for advice.

Determination of particle size

Fill the INQUA[®] nebuliser with 3 ml of a 2.5% test solution and then attach the INQUA[®] nebuliser to an impactor (NGI[™] = Next Generation Impactor[™]) and run for 3 minutes.

Cumulative mass distribution for the INQUA[®] nebuliser at the minimum flow (3.0 l/min) and the maximum flow (6.0 l/min):

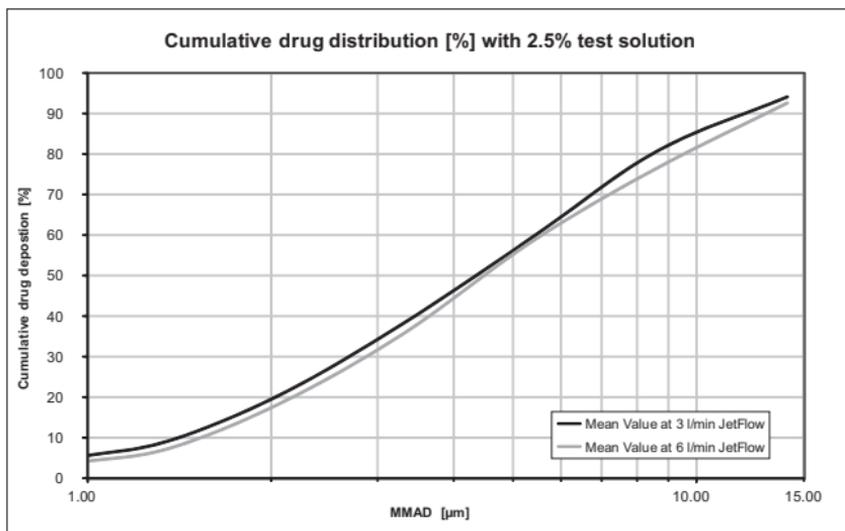(Averages from each of 2 separate measurement series with 3 nebuliser/compressed air combinations)

Determination of aerosol output²⁾

Fill the nebuliser with 2 ml of a 1% test solution.

Attach the nebuliser to the Breath Simulator until the aerosol becomes irregular, plus one more minute.

Aerosol output in [ml] for minimum flow (3.0 l/min) and maximum flow (6.0 l/min):

Min. flow (3.0 l/min)	0.21 ml
Max. flow (6.0 l/min)	0.19 ml

(Averages from each of 2 separate measurement series with 3 nebuliser/compressed air combinations)

Determination of the aerosol output rate³⁾

Fill the nebuliser with 2 ml of a 1% test solution. Attach the nebuliser to the Breath Simulator for at least 1 minute.

Aerosol output rate in [ml/min] for minimum flow (3.0 l/min) and maximum flow (6.0 l/min):

Min. flow (3.0 l/min)	0.096 ml/min
Max. flow (6.0 l/min)	0.144 ml/min

(Averages from each of 2 separate measurement series with 3 nebuliser/compressed air combinations)

We reserve the right to make technical changes.

Information as of: 2019-05

- 2) Aerosol quantity output by the nebuliser system for a specified fill volume*
- 3) Quantity of aerosol that is output by the nebuliser system per unit of time*

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