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## belAir® NaCl 0,9 %

zur Inhalation

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## belAir® NaCl 0,9 %

for inhalation

**Contents:** sterile, preservative-free isotonic saline solution (0.9% NaCl) per ampoule

### Areas of use and intended use:

- Do not use after the expiry date.
- Once opened, ampoules must be used completely, do not store opened ampoules. Bacteria can breed very quickly in the liquid in open ampoules, representing a risk to health, for example by causing infections.
- People with mental, physical or sensory disabilities and children do not recognise the risks that may arise when using the ampoules and their components. Never allow anyone from these groups access to the ampoules without supervision.
- Small parts can be swallowed and block the respiratory tract, thus posing a choking hazard. Therefore, keep and dispose of full and empty ampoules out of the reach of children and all persons with mental, physical or sensory disabilities.
- Danger to health from improper use.
- Read and follow the information in the instructions for use of the nebuliser and the inhalation device as well as for the medication to be inhaled.

Sterile belAir® NaCl 0,9 % is used:

- to moisten the upper and lower respiratory tract as a treatment for cold-like symptoms such as coughing or nasal congestion by nebulisation with an electric jet, ultra-sound, or membrane nebuliser.
- to dilute medications for inhalation.

The medication must have been designed for this administration form.

In this regard, read and follow the instructions for use of the medication and seek advice from your doctor or pharmacist.

belAir® NaCl 0,9 % is suitable for use by persons of all ages. Accordingly, it may even be administered to infants.

### Application:

- Open the ampoule.
- To dilute medication, follow the appropriate instructions for use.
- Fill the nebuliser as directed in the instructions for use of the inhalation device.
- Inhale as described in the instructions for use of the inhalation device you are using.

The duration of an inhalation session may vary, but should not exceed 20 minutes for 3 ml of belAir® NaCl 0,9 %. If it does, please contact the manufacturer of your inhalation device.

Isotonic saline solution may cause constriction of the bronchia when inhaled by very sensitive patients such as those suffering from bronchial asthma or who have a hyperreactive bronchial system; this constriction is reversible. If this does occur, stop treatment and seek advice from your doctor or pharmacist.

**Application temperature:** Room temperature

**Dosage:** Contents of one ampoule of belAir® NaCl 0,9 % as needed up to a maximum of 4 times a day

### Administration form:

502G0350: 30 x 3 ml ampoules

### Manufacturer:

INQUA GmbH  
Moosdorfstr. 1  
82229 Seefeld  
Germany

Tel.: +49 (0)8152-9934 0  
Fax: +49 (0)8152-9934 20  
E-mail: info@bel-air.de  
www.bel-air.de

### Explanation of symbols:

	Manufacturer		Lot no.
	Warning		Use by
	Sterile		Please follow the instructions for use
	Do not reuse		Order number

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